Amplified Telephone (40 dB) with Talking-Keypad,
Speakerphone & Voice-Activated-Answering

Operating Guide

Thank you for purchasing this
product, please read before use
(Keep this manual for future reference)

HD-50JV

HDS
TECHNOLOGY

CONTENT

1. Product features	3	17. Dialing from a Two-touch memory	10
2. Important safety instructions	4	18. Call Waiting	10
3. Battery and AC Power Adapter	5/6	19. Redial	10
4. Package content	6	20. Pause Function	10
5. Installation	6	21. Set Talking Keypad	11
6. Detailed features and controls	7	22. Set Voice Activated Answering	11
7. Answering A Call	8	23. Set Language	11
8. Answer a Call by Voice Activated Answering	8	24. Set Phone Number Announcing	12
9. Making A Call	8	25. Ringer Volume	12
10. Transferring a Call from Handset to Speakerphone	8	26. Tone or Pulse Dialing Mode	12
11. Volume Control (Receiver and Speakerphone)	8	27. Mixed Dialing (Tone and Pulse Dialing)	12
12. Amplify Key and Receiver Tone Control	9	28. Technical Information	13
13. Auto "Amplify" Switch	9	29. Trouble shooting	13
14. Programming a One-Touch Memory	9	30. FCC & IC Registration Information	14
15. Dialing from a One-Touch Memory	9	31. Warranty Service Information	15
16. Programming a Two-Touch Memory	10		

PRODUCT FEATURES

1. High Definition Sound Technology ensures crystal clear conversation
2. Up to 40 dB Of handset amplification
3. “No-touch voice activated answering” on incoming calls
4. Talking keypad & controls
5. “AMPLIFY” key that remembers handset volume setting
6. Four (4) digital tone control settings
7. High performance speakerphone
8. Selectable talking language (English, Francais, & Espanol)
9. Three one-touch memories
10. Ten 2-touch memories
11. Super loud ringer
12. Dual ring flasher (front and side)
13. Hold with indicator
14. Battery back-up option on Base

Important Safety Instructions

Instructions pertaining to a risk of fire, electric shock or injury to persons.
Privacy of communications may not be ensured when using a cordless phone.

WARNING – When using electric appliances, basic precautions should always be followed, including the following:

1. Read all the instructions before using the appliance.
2. This product uses rechargeable batteries. Do not use any non-rechargeable battery as charging of this type can cause battery leakage.
3. Always disconnect phone cord to wall jack before changing or installing batteries.
4. To reduce the risk of injury, close supervision is necessary when an appliance is used near children.
5. Do not contact moving parts or any exposed metal surface.
6. Only use attachments recommended or sold by the manufacturer.
7. Do not use outdoors. Do not use near water.
8. To disconnect, turn all controls to the off position, then remove plug from outlet.
9. Do not unplug by pulling on cord. To unplug, grasp the plug, not the cord.
10. Unplug from outlet when not in use and before servicing or cleaning.
11. Do not operate any appliance with a damaged cord or plug, or after the appliance malfunctions or is dropped or damaged in any manner. Return appliance to the nearest authorized service facility for examination, repair, or electrical or mechanical adjustment.
12. To Reduce the Risk of Electric Shock, this appliance has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.
13. Avoid using any telephone product during an electrical storm. There may be a remote risk of electric shock from lightning.

14. Do not use the telephone to report a gas leak in the vicinity of the leak.
15. Specifications of this product are subjected to change without notice.

Optional Batteries for Base

Base: Accepts four (4) AA size regular batteries (user provided) for AC power outage backup purpose.
Installation of base batteries is optional.

Please care for the environment by disposing any used batteries through your local community recycle center.
Do not dispose any used battery in trash bin.

WARNING: For safety consideration, always disconnect phone cord to wall jack before changing batteries

Disposal of Used Batteries and Safety Information

1. Do not dispose of the battery in a fire as it may explode.
Please dispose any used battery through your local community recycle center.
2. Do not open or mutilate the battery. Released electrolyte is highly corrosive and may cause damage to the eyes and skin.
It may be toxic if swallowed.
3. Exercise care in handling batteries in order not to short the battery with conducting material such as rings, bracelets and keys. Shorting a battery may overheat and cause burns.


AC Power Adapter

For your safety, please plug the provided AC power adapter into the nearest AC outlet. Use only UL listed Class 2 power adapter, 9VDC, 300 MA.


Package Content

Base Unit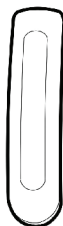

Handset

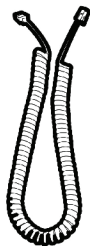Coil Cord


Line cord

AC Adapter


Wall mount bracket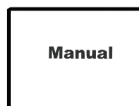


User' s Manual

Installation

1. Connect AC adapter and phone cord to the base as shown.
2. Install optional batteries in base. These batteries ensures full handset amplification and speakerphone function in case of power outage (for your safety, always disconnect phone cord to wall before changing / installing batteries). Otherwise, the phone reverts to a simple phone (no amplification) during power outage.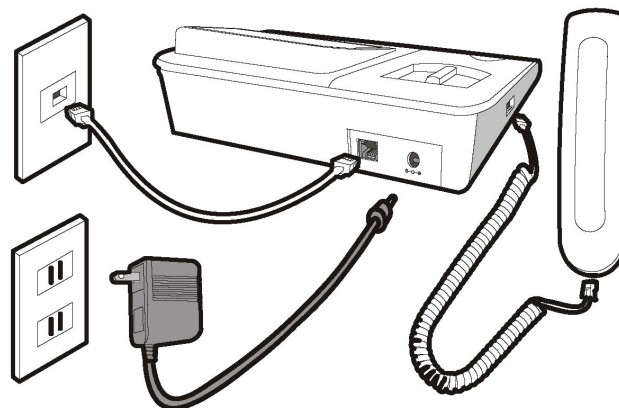

Detailed Features and Controls

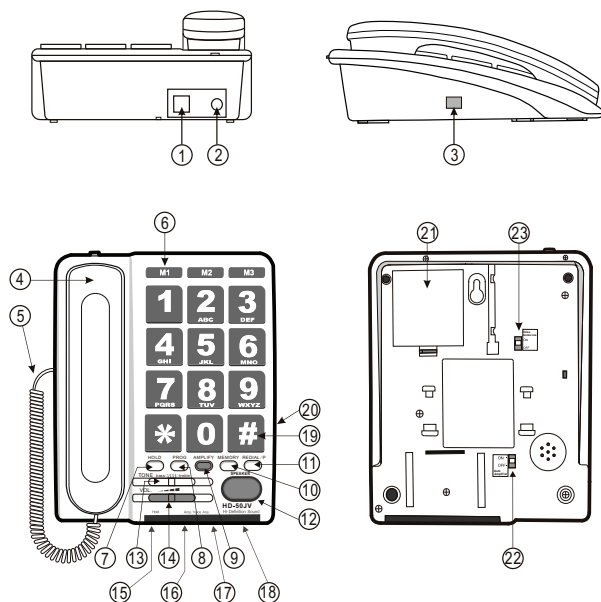1. Phone Jack
2. AC adapter jack
3. Handset jack
4. Handset
5. handset cord
6. One-touch memory (M1)
7. "HOLD" key
8. "PROGRAM" key
9. "AMPLIFY" key
10. "MEMORY" key
(for Two-touch memories)
11. "REDIAL / PAUSE" button
12. "SPEAKERPHONE" key with
indicator
13. Digital tone selector
14. Volume control (for handset
and speakerphone)
15. "HOLD" indicator
16. Low battery indicator
17. "AMPLIFY / VOICE
ANSWER" indicator
18. Ringer flasher (front and side)
19. Keypad
20. Ringer Loudness
(Hi-Med-Lo)
21. Battery compartment
22. Auto Amplify On-Off control
23. Talking keypad On / Off
(Voice Announce On / Off)

Answering a Call

1. Lift handset to answer (or press  key to use speakerphone) when phone rings.
2. Hang up handset (or press  key again) to end the call.

Answer a Call by Voice Activated Answering

Note: This function allows a user to answer a call without touching the phone. It is enabled via menu selection. When enabled, the “Amplify / Voice Answer” indicator flashes periodically under the red windows in front.

1. Stay within a 5 feet radius of the phone when it rings, say “H-E-L-L-O” aloud during the silent interval of ringing activates the speakerphone and answers the call. In fact any loud voice or sound longer than 1-second in duration would work. Always wait 0.5 second into the silent interval before answering by voice.
2. Press  key to end the call.

Making a Call

1. Lift handset to dial (or press  key) to turn on speakerphone and dial.
2. Hang up handset (or press  key again) to end the call.

Transferring a Call from Handset to Speakerphone

1. While on a call using the handset, press the  key and then hang up the handset.
2. The conversation is now transferred to the speakerphone for hands-free conversation. (Lifting the handset again turns off the speakerphone).
3. Press  key again to end the call.

Volume Control (receiver and speakerphone)

1. While on a call, slide volume control  to adjust.
2. Hang up handset (or press  key again) to end the call.

Amplify Key and Receiver Tone Control

1. While on a call using handset, press **AMPLIFY** key to add an extra level of amplification (25 dB) to the receiver volume. This is indicated by the “Amplify” indicator under the front window.
2. While the **AMPLIFY** key is active, the  control also becomes active. Experiment different tone and volume settings for best voice clarity. As a rule, use the same tone setting on future calls. (Note: Tone control does not work in speakerphone).
3. Hang up handset to end the call.

Programming a One-Touch Memory

1. While off-line, press **PROG** key and then enter phone number.
2. Press a One-Touch memory key to save.
3. Repeat Steps 1 and 2 for other One-Touch memories.

Auto “Amplify” Switch

This switch is located on the bottom of the phone, and it determines the initial condition of the AMPLIFY function. If this switch is set to ON, the **AMPLIFY** key will be active each time you lift the handset off its cradle. (When this key is activated, an extra level of amplification is added to the receiver volume and tone control also becomes active). This setting is recommended if this phone is only used by a single user. If this switch is set to OFF, the user must manually press the **AMPLIFY** key in order to activate the extra receiver volume. This setting is recommended if this phone is share by two or more users.

Dialing from a One-Touch Memory

1. Lift handset, press a one-touch memory key to select. There will be audio confirmation if the “Talk-back” feature is enabled.
2. Hang up handset to end the call.

Programming Two-Touch Memories

1. While off-line, press **PROG** key and then enter phone number.
2. Press **PROG** again, and then press “1” on the keypad to save into a Two-Touch memory “A-1”.
3. Repeat Steps 1 and 2 for other Two-Touch memories (A-2 to A-0).

Dialing from a Two-Touch Memory

1. Lift handset, press **MEMORY** and then an appropriate key from “1” to “0” to select a Two-Touch memory. There will be audio confirmation if the “Talking Keypad” feature is on.
2. Hang up handset to end the call.

Redial

1. Lift the handset to go on-line and then press **REDIAL**. (Or press **SPEAKER** key to go on-line and then press **REDIAL**).
2. Hang up the handset to end the call.

Call Waiting

While talking on the phone, a soft beep tone in the receiver indicates a new call is waiting. You may answer this new call by momentary press down on the hook switch and then let go. This places the original caller on “hold” while you are talking to the new caller.

At the end of the conversation, momentary press down on the hook switch again reconnects you back to the original caller.

Pause Function

During one-touch (or two-touch) memory programming, press **REDIAL/P** key inserts a 3-second pause into the dialing sequence. For example: 9 P 7 2 2 8 8 5 0 where “P” represents a 3-second pause. This pause is needed to access an outside line on some business phone systems.

Set Talking Keypad (Voice Announce) On / Off

This function enables the keypad Talk-back function. When set to on, each press of the keypad or function key provides voice feedback.

1. On the bottom of the phone base, slide “Voice Announce” control to ON or OFF accordingly.
2. When it is On, it will announce the name of the key you press.

Set Language (English, Francais, Espanol)

1. While off-line, press and hold **PROG** key for about 3 second to enter function setting mode. You hear a voice confirmation “Program, Program”.
2. Press “ * * “ to hear current language setting.
3. Press “#” key to change language setting. Each press of “#” key toggles to the next language setting.
4. Press **PROG** again to save the new language setting and then exit. An “OK” announcement confirms exit to off-line mode.

Set Voice Activated Answering

When this function is set to ON, a user can answer an incoming call by voice. The “Amp / Voice Ans” indicator in the front window flashes periodically when this feature is activated. (see Page 8).

1. While off-line, press and hold **PROG** key for about 3 second to enter function setting mode. You hear a voice confirmation “Program, Program”.
2. Press “ # # “ to hear current voice answering setting.
3. Press “#” key to toggle the setting from ON to OFF or vice-versa.
4. Press **PROG** again to save the new setting and then exit. An “OK” announcement confirms exit to off-line mode.

Set Phone Number Announcing On / Off

When dialing from a memory, both the memory name and its phone number will be announced. However, the phone number portion may be deleted so only the memory name is announced.

1. While off-line, press and hold **PROG** key for about 3 second to enter function setting mode. You hear a voice confirmation "Program, Program".
2. Press "# 3" to hear current phone number announcing setting (On / Off).
3. Press "#" to toggle this setting.
4. Press **PROG** again to save new setting and then exit. An "OK" announcement confirms exit to off-line mode.

Ringer Volume

A three position ringer loudness switch is located on the side of the phone.

Mixed Dialing (Pulse and Tone dialing)

Certain applications, like banking or voice mail retrieval, require the use of DTMF tone while the phone is set for pulse dialing. After pulse dialing, pressing "*" key temporary changes the keypad into tone mode for banking or voice mail retrieval. It will automatically return to pulse dialing on the next call.

Tone or Pulse Dialing Mode

1. While off-line, press and hold **PROG** key for about 3 seconds to enter function setting mode. You hear a voice confirmation "Program, Program". (Assume "Voice Announce" function is OFF).
2. Press "# 1" to hear current dialing mode setting as either "1" or "2" (1 = Tone, 2 = Pulse).
3. Press "#" to toggle this setting.
4. Press **PROG** again to save new setting and then exit. An "OK" announcement confirms exit to off-line mode.

Technical Information

Amplified dB level (Amplify off): 0 to 15 dB
Amplified dB level (Amplify on): 25 to 40 dB
Tone control settings: 4 settings (T1 to T4)
Talking Keypad voice: English, Francais, Espanol)

Specifications are subjected to change without any notice.

Dimensions

Handset size: 7.0" x 2.1" x 1.2"
Base size: 6.7" x 3.1" x 5.0"

Weight

Handset weight: 5.7 oz.
Base weight: 16 oz.

Temperature range:

Operating: 0 to 30°C
Storage: -10 to 40°C

Power requirements:

AC Adapter: UL listed 9 vdc, 300 ma
Base battery: Optional 4 x AA size regular batteries
(provided by user).

Trouble Shooting

Q1: *How come there is no dial tone?*

Ans: Please verify both phone cord and the handset cord are plugged in properly. Verify the AC adapter is also plugged in properly.

Q2: *Why do I need to put batteries in my phone?*

Ans: Batteries are optional. They provide back up during a power outage situation. If there are no batteries, the phone will act like a simple phone and no amplification during power outage.

Q3: *Why is there Static noise on the line?*

Ans: Your phone may be located near a touch lamp, microwave, or other appliance. If you have a DSL computer modem service, make sure you have installed a DSL filter to block the DSL noise.

Q4: *Why did the phone answer the call by itself?*

Ans: The phone is set to "Voice activated answering". Please follow instructions on Page 8 to turn off this feature.

FCC Information for USA Customers only

1. This equipment complies with Part 68 of the FCC rules and the requirements adopted by the ACTA. On the bottom of this equipment is a label that contains, among other information, a product identifier of [US: XXXXXXXXX]. If requested, this number must be provided to the telephone company.
 2. If this equipment, HD-50JV, causes harm to the telephone network, the telephone company will notify you in advance that temporary discontinuance of service may be required. But if advance notice isn't practical, the telephone company will notify the customer as soon as possible. Also, you will be advised of your right to file a complaint with the FCC if you believe it is necessary.
 3. The telephone company may make changes in this facilities, equipment, operations or procedures that could affect the operation of the equipment. If this happens the telephone company will provide advance notice in order for you to make necessary modification to maintain uninterrupted service.
 4. If you experience trouble with this equipment, you disconnect it from the network until the problem has been corrected or until you are sure that the equipment is not malfunctioning.
 5. Please follow instructions for repairing if any (e.g. battery replacement section); otherwise do not alternate or repair any parts of device except specified.
 6. Connection to party line service is subject to state tariffs. Contact the state public utility commission public service commission or corporation commission for information.
 7. If the telephone company requests information on what equipment is connected to their lines, inform them of:
 - A. The telephone number that this unit is connected to
 - B. The ringer equivalence number [REN]
 - C. The USOC jack required [RJ11C], and
 - D. The FCC Registration Number [US: XXXXXXXXX]
- E. Items (B) and (D) are indicated on the label. The ringer equivalence number (REN) is used to determine how many devices can be connected to your telephone line. In most areas, the sum of the RENs of all devices on any one line should not exceed five (5.0). If too many devices are attached, they may not ring properly.

Warranty Service Information

Your HD Phone comes with a one-year limited warranty. We warrant during this warranty period, from the date of purchase, to the original consumer, the HD Phone to be free from defects in materials and workmanship under intended home use. In the event that HD Phone fails to function properly within one year of original purchase due to defects in materials or workmanship, return the unit (freight prepaid) with proof of purchase (sales receipt or packing slip, no exceptions) to Serene Innovations. Serene Innovations will either repair or replace the unit (with a refurbished unit or unit of equal condition) and return it to you (using UPS / USPS ground shipping) at no cost to you if the unit is returned within 30 days of purchase. If the unit is returned after 30 days of purchase but within the warranty period, there will be a warranty handling charge of \$9.95 for each return.

Incidental or Consequential Damages: Neither Serene Innovations, Inc. nor retailer, dealer or selling distributor has any responsibility

for any incidental or consequential damages including without limitation, commercial loss of profit, or for any incidental expense, loss of time, or inconvenience. Some states do not allow exclusion or limitation of incidental or consequential damage, so the above limitation or exclusion may not apply to you.

This warranty does not apply to any product that has been accidentally damaged, abused, misused or negligent, used on electrical frequency or voltage other than marked on product and/or described in manual. Defects or errors caused by unauthorized alterations, repairs, and/or tampering are also not covered by this warranty.

This Warranty gives you specific legal rights, and you may also have other legal rights that vary from states to states.

If a defect covered by this warranty should occur, promptly contact a Customer Care Representative by phone or log onto [www.](http://www.sereneinnovations.com)

sereneinnovations.com to obtain a Return Authorization Number (RAN) and shipping instructions before shipping the product to us. Any shipment without a RAN will not be accepted and will be returned to you at your expense. Any authorized returned product must be accompanied with proof of purchase and a brief explanation of the problem.

For out of warranty repaired and service, please contact our Customer Care Department for directions.

This equipment complies with Part 68 of the FCC Rules. On the equipment is a label that contains, among other information, the FCC registration number and ringer equivalence number (REN) for this equipment. If requested; this information must be provided to your telephone company.

This equipment uses the following Universal Service Order Codes ("USOC") jacks: RJ-11. Connection to the telephone network should

be made by using standard modular telephone jack type RJ11.

The REN is useful to determine the quantity of devices you may connect to your telephone line and still have all of those devices ring when your telephone number is called. In most, but not all areas, the sum of the RENs of all devices connected to one line should not exceed five (5.0). To be certain of the number of devices you may connect to your line, as determined by the REN, you should contact your local telephone company to determine the maximum REN for your calling area.

If your telephone equipment causes harm to the telephone network, the telephone company may discontinue your service temporarily. If possible, they will notify you in advance. But if advance notice is not practical, you will be notified as soon as possible. You will be informed of your right to file a complaint with the FCC.

Your telephone company may make changes in its facilities, equipment, operations or procedures that could affect the proper functioning of your equipment. If they do,

you will be notified in advance to give you an opportunity to maintain uninterrupted telephone service.

In the event this equipment should fail to operate properly, disconnect the unit from the telephone line. Try using another FCC approved telephone in the same telephone jack. If the trouble persists, call the telephone company repair service bureau.

If the trouble does not persist and appears to be with this unit, disconnect the unit from the telephone line and discontinue use of the unit until it is repaired. For repair or warranty information, please contact Serene Innovations, Inc. at 562- 407 5400. Please note that the telephone company may ask that you disconnect this equipment from the telephone network until the problem has been corrected or until you're sure that the equipment is not malfunctioning.

There are no user serviceable parts in this equipment.

This equipment may not be used on coin service provided by the telephone company. Connection to party lines is subject to state tariffs. Contact the state public utility commission, public service commission for information.

If your home has specialty wired alarm equipment connected to the telephone line, ensure the installation of this equipment does not disable your alarm equipment. If you have questions about what will disable alarm equipment, consult your telephone company or a qualified installer.

Part 15 of FCC Rules Information

This device complies with Part 15 of the FCC rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may caused undesired operation.

This equipment has been tested and found to comply with the limits of a Class B digital device, pursuant to Part 15 of FCC rules. These limits are designed to provide

reasonable protection against harmful interference in residential installation.

This equipment generates and can radiate radio frequency energy, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, you may try one the following suggestions to correct the interference:

1. Where it can be done safely, re-orient the receiving television or radio antenna.
2. To the extend possible, relocate the television, radio, or other receiver with respect to the cordless phone.
3. Connect the cordless phone AC adapter into a different electrical outlet.
4. Consult the dealer or an experienced radio/TV technician for help.

Modification: Any modification not expressly approved by the manufacturer of this device could void the user's authority to operate the device.

NOTICE: The Industry Canada label identifies certified equipment.

This certification means that the equipment meets telecommunications network protective, operational and safety requirements as prescribed in the appropriate Terminal Equipment Technical Requirements document(s). The Department does not guarantee the equipment will operate to the user's satisfaction.

Before installing this equipment, users should ensure that it is permissible to be connected to the facilities of the local telecommunications company. The equipment must also be installed using an acceptable method of connection. The customer should be aware that compliance with the above conditions may not prevent degradation of service in some situations.

Repairs to certified equipment should be coordinated by a representative designated by the supplier. Any repairs or alterations made by the user to this equipment, or equipment malfunctions, may give the telecommunications company cause to request the user to

disconnect the equipment. Users should ensure for their own protection that the electrical ground connections of the power utility, telephone lines and internal metallic water pipe system, if present, are connected together. This precaution may be particularly important in rural areas.

Caution: Users should not attempt to make such connections themselves, but should contact the appropriate electric inspections authority, or electrician, as appropriate. The REN for this amplified telephone as stated on the IC regulatory label located on the bottom of the product.

NOTICE: The Ringer Equivalence Number (REN) assigned to each terminal device provides an indication of the maximum number of terminals allowed to be connected to a telephone interface. The termination on an interface may consist of any combination of devices subject only to the requirement that the sum of the Ringer Equivalence Numbers of all the devices does not exceed 5. This Class B digital device complies with Canadian ICES-003.

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Operating Guide

Thank you for purchasing this product, please read before use
(Keep this manual for future reference)

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